

REPLACEMENT DESICCANT INSTRUCTIONS FOR SMALL CUSTOM LABEL DEHUMIDIFIER

item# 0201006

1. The granules should be used with the drying jar.
2. Open the drying jar, remove the drying disc from the foil and put the drying disc in the jar.
3. Open the battery door, lay down your hearing aid and shut the lid. Keep jar lid closed.
4. When granules turn **white** it is time to recharge. Place drying disc in microwave for 30 seconds on high.

IMPORTANT:

- **Keep out of reach of children.**
- **Do not eat.**
- **Do not put hearing aid in microwave.**

