

Non-Surgical Isolation Gowns (T Gown)

SOLUTIONS FOR MEDICAL PROVIDERS AND FIRST RESPONDERS



For removal, tear perforation on back side of gown

These polyethylene, non-surgical isolation gowns are easy to put on (don) and take off (doff). The full length, long sleeve style provides full coverage. The sleeves include a thumb loop to secure the sleeve.

Under AAMI Guidelines, protective apparel must meet minimum performance standards for strength, barrier protection and fluid resistance to ensure the safety of healthcare workers. These gowns have been tested for compliance with ANSI/AAMI PB70:2012 Minimal Barrier Performance Level 2 requirements.

FEATURES

- Tested for compliance with ANSI/AAMI PB70:2012 Minimal Barrier Performance Level 1 and Level 2 requirements
- Easy to put on and take off
- Back side is marked and perforated for easy removal
- Openings in end of sleeve seal for fingers/hand
- Long sleeve, full coverage
- Made from polyethylene (Virgin LLDPE)

(Donning/Doffing instructions on reverse)

WARNING

- This product is **NOT** sterilized.
- This product is for **single use ONLY**.
- Do **NOT** use in the presence of any high intensity heat source or flammable gas.

SKU	Model	Neck	Mil	Film Color	Material	Style	Span	Length	Case Pack	Cases/Pallet
TgownLP	74	12"	1.5	Blue	Polyethylene	Group Folded	74"	50"	50	84

Manufactured in USA / Patent Pending


For more information contact your Heritage Sales Representative

Follow this link for more information about preparing the gown for use



Non-Surgical Isolation Gowns

DONNING AND DOFFING INSTRUCTIONS

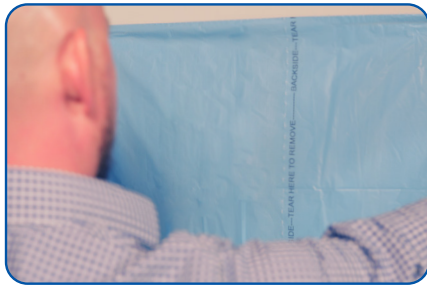

1

Locate the back of the gown, with the print facing the user.

Break the perforation line below each sleeve to separate the sleeves from the body of the gown.

2



3

Open the gown from the bottom, and insert hands into the sleeves.

Push thumbs and fingers through the perforated openings at the end of each sleeve.

4



5

Gather the gown together at the neck opening and slip the entire gown over the head.

Wrap the extra body material to the back side and tie or use an adequate amount of tape to secure.

6



7

Put on gloves according to CDC guidelines.

For easy removal, tear along the vertical perforation line on the back.
Do off the gown per CDC guidelines.