



The ICS Impulse Vision Denied Solution 2 Kit

Create a more comfortable experience for your patients and enhance your workflow efficiency

The Vision Denied Solution 2 Kit for your ICS Impulse that features a face cushion and goggle cover, is designed to deny and bring back patient fixation throughout testing without removing the ICS Impulse from the patient's head.

Ready to order!



Enhance the patient experience

The Vision Denied Solution 2 Kit makes testing more comfortable by eliminating the need to repeatedly apply and remove the ICS Impulse goggles from the patient's head during testing with Otosuite® Vestibular Software workflows.

Simply attach the single-use face cushion provided inside each kit to the inside of ICS Impulse before use and apply the ICS Impulse to the patient's head to create a comfortable and secure fit for the duration of testing workflows.

During testing, when patient's vision-denied fixation is required, apply the Vision Denied Solution 2 goggle cover to the outside of ICS Impulse to deny the patient's eyes vision. To bring back the patient's fixation, remove the goggle cover.

When the test workflows are complete, easily remove the goggle cover during the testing workflows where vision fixation is required. Reduce the number of steps made during testing and create a more pleasant experience for the patient, while improving accuracy, and reducing overall testing workflow times with the Vision Denied Solution 2 Kit.

Simplify your workflow

- Use goggle cover with the ICS Impulse and the face cushion in-situ
- Track eye movements in the vision and vision-denied state
- Continue to assess, record and measure the vestibulo-ocular reflex and nystagmus while in the vision-denied state
- Increase the efficiency of mVNG by reducing the number of steps to speed up workflow
- Does not require recalibration

Equipment

The Vision Denied Solution 2 Kit includes the Face Cushion DV (Denied Vision) and a Goggle Cover, and is designed for use with the ICS Impulse goggles.

The Vision Denied Solution 2 Kit is a single-use product and available in case batches of 50 pcs (PN: 041481) or 200 pcs (PN: 041482).



Want to buy the new ICS Impulse Vision Denied Solution 2 Kit? Contact your local Natus representative today and learn how to order!

Learn more about ICS Impulse at natus.com/impulse

Natus Medical Incorporated is an audiology industry leader. We help hearing and balance care professionals improve the quality of life for millions of people around the globe. Natus improves patient outcomes using the most comprehensive product solutions, depth of training and continuing customer care when diagnosing and treating disorders of the central nervous and sensory systems. For more information, please visit natus.com.

Making sense of the body's signals

©2023 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. **044671 RevA**

natus®

Natus Medical Incorporated

natus.com