## **OtoClear® Adapter Wand Instructions**

For Use with Portable Waterpik® (Reorder #7215)



1. Push the adapter wand into the positioning wheel.



2. Twist the *OtoClear*® tip on the luer lock fitting of the adapter wand. (Be sure to tighten the *OtoClear*® tip securely).

NOTE: The wand can be cleaned as needed depending on usage rate.

Use a soft cloth and mild, non-abrasive cleaner.



Safe Irrigation Tip is not affiliated with Waterpik

ofWater Pik, Inc. The OtoClear Technologies.© 2015 Bionix De