This Professional PenLite is for general, NON-CONTACT illumination only.

CAUTION: Do not use for transillumination as burns could result. This instrument is not to be used in the presence of flammable anesthetics.

Properly dispose of batteries according to national or local regulations.



CHANGING THE LAMP:

1. Unscrew lamp/tip assembly.
Discard. Replace with Welch
Allyn #07600 lamp/tip
assembly.

- CHANGING BATTERIES:1. Unscrew lamp/tip assembly and remove old batteries.2. Insert new AAA batteries (may be inserted either
- way). For best light output and battery life use alkaline batteries.3. Replace lamp/tip assembly.