**A-02**  
**TV Bundle**

AUDEARA

<b>ENGLISH</b>	
A-02 TV BUNDLE GUIDE .....	5
<b>DEUTSCH</b>	
A-02 TV-PAKET – ANLEITUNG.....	33
<b>ESPAÑOL</b>	
GUÍA DEL LOTE A-02 TV.....	61
<b>FRANÇAIS</b>	
A-02 GUIDE DES BOUQUETS TV .....	89
<b>ITALIANO</b>	
GUIDA AL PACCHETTO TV A-02.....	117
<b>MAGYAR</b>	
A-02 TV CSOMAG, HASZNÁLATI ÚTMUTATÓ .....	145
<b>NEDERLANDS</b>	
A-02 TV-BUNDELGIDS.....	173
<b>POLSKI</b>	
PRZEWODNIK PO ZESTAWIE TELEWIZYJNYM A-02 .....	201

**ENGLISH**  
**A-02 TV BUNDLE GUIDE**

# Contents

## OVERVIEW

What's in the box	6
Headphones overview	8
TV Streamer overview	10

## HEADPHONES

Connect your Headphones	13
Tailor your Headphones	17
Headphones tips	18

## TV STREAMER

Connect your TV Streamer	21
Connect to additional devices	28

# What's in the box

Headphones

Carry Case


TV Streamer


Accessory Bag

## HEADPHONES ACCESSORIES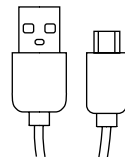

USB-C  
Power Cable

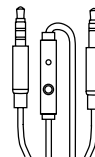
3.5mm Audio Cable  
(with microphone)


Carabiner

## TV STREAMER ACCESSORIES

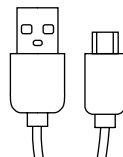
USB-C  
Power Cable

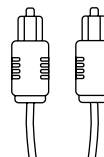Digital Optical  
Audio Cable


Analog  
Audio Cable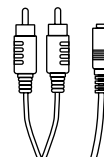


AV Adapter  
Cable

# Headphones


Left Ear Cup  
Headphone Controls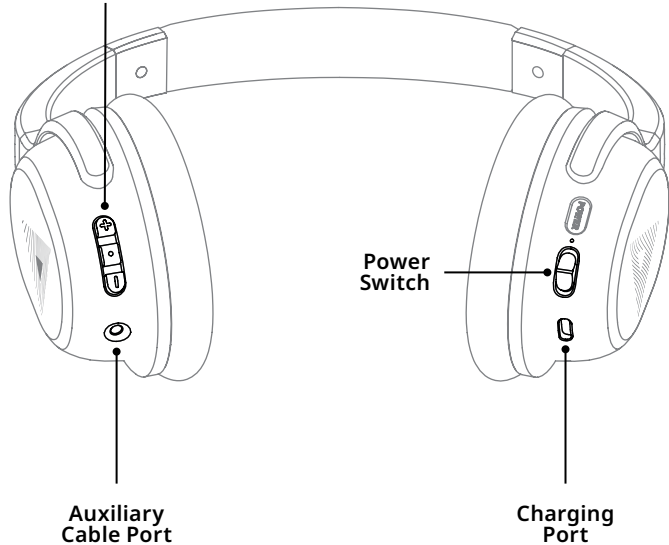

LEFT EAR CUP

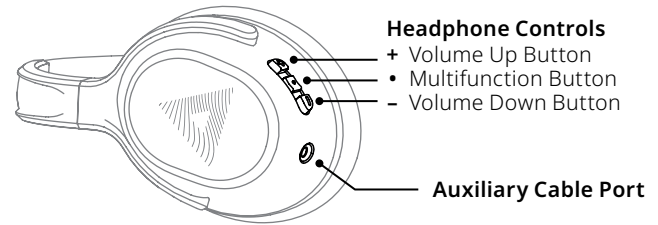
RIGHT EAR CUP

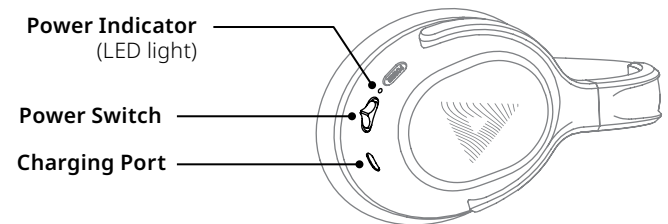
# TV Streamer

## FRONT VIEW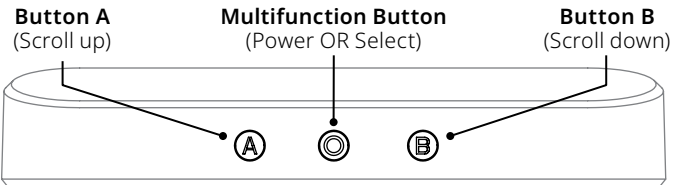

## BACK VIEW

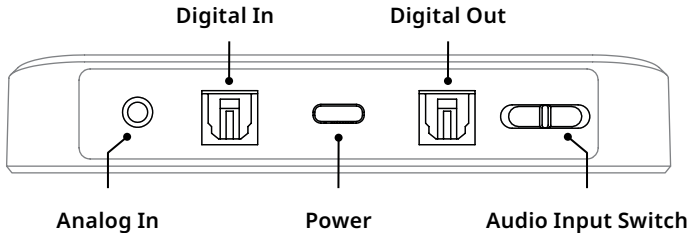
## TOP VIEW

The below display areas will appear on screen when powered on.

### Audio input selection

- ▶ Analog (AUX)
- ▶ Digital (OPT)

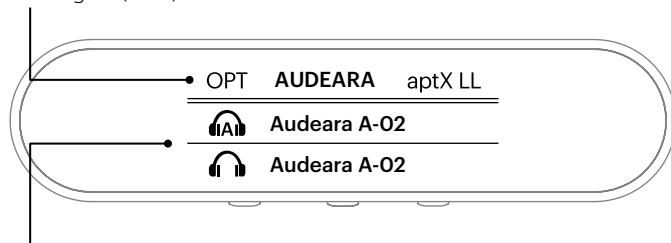### Connected device(s)

A list of connected devices will appear here when paired. The headphone icon will change to indicate if the device is connected:

Paired and connected

Paired and disconnected

# Connect your Headphones

Congratulations for putting your hearing health first by choosing Audeara. We're excited to help you feel connected and rediscover the joy of listening.

Your Audeara Headphones can connect to any Bluetooth® enabled device. This allows you to enjoy music and make audio and video calls wirelessly.


## 1. Place your Headphones in Pairing Mode

Power your Headphones on by flicking the Power Switch up

- ▶ This will make the Power Indicator flash blue.
- ▶ Your Headphones will also provide an audible prompt that says “Power On”.

Hold down the Multifunction (middle) Button for three seconds

- ▶ This will make the Power Indicator flash blue and red.
- ▶ When in pairing mode, your Headphones will also provide an audible prompt that says “Pairing” followed by a repeated chime while in pairing mode.

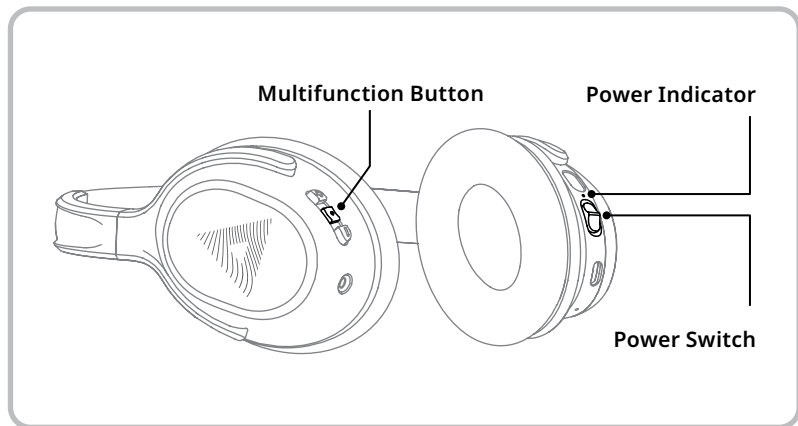## 2. Connect to a mobile device

On your mobile device, navigate to Settings > Bluetooth and ensure Bluetooth is turned on

- ▶ You should see “Audeara A-02” in the device list. If it doesn’t appear, see step 1.

Tap on “Audeara A-02” to pair to your device

- ▶ Once the device is connected, your Headphones will stop flashing blue and red and you will also hear an audible prompt that says “Connected”.

## 3. Connect to a second mobile device (optional)

Your Headphones can pair to two devices at the same time

- ▶ With your Headphones already connected to the first device, repeat steps 1 and 2 with the second device.
- ▶ Turn your Headphones off and then on again using the Power Switch to connect to both paired devices.

# Experience sound tailored to you

Now that you've connected your Headphones to your mobile device, you can tailor your Headphones to your hearing profile with the Audeara app.

We all hear differently - even your left ear hears differently to your right. Connect your Audeara Headphones to our app and take a simple hearing health check in just minutes.

Use your results to tailor your Audeara Headphones to your individual hearing profile.


Once tailored, your Headphones store your hearing profile regardless of what device you connect to. You can change your profile at any time using the Audeara app.

Scan to  
Download

# Headphone tips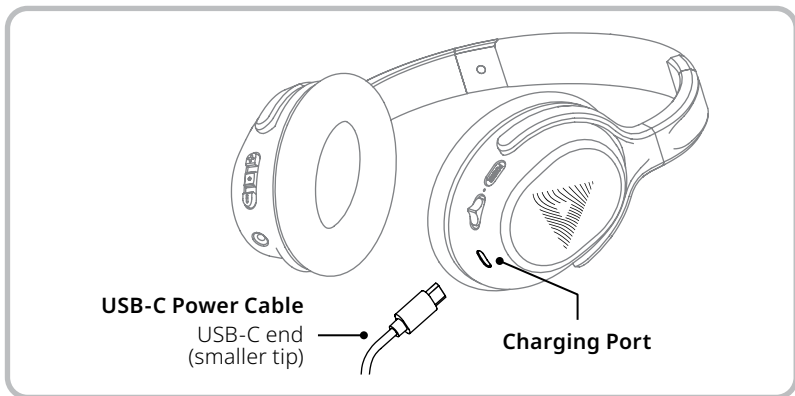

## POWER INDICATOR

**Red LED** indicates that the Headphones are charging.

**Blue LED** indicates that the Headphones are charged.

## Charging your Headphones

Your Headphones have a battery life of 30+ hours

- ▶ Use the USB-C Power Cable provided to charge your Headphones.
- ▶ Insert the USB-C end (smaller tip) into the port on the right earcup of the Headphones.
- ▶ Insert the USB-A (bigger tip) into a USB power source. At this stage you will see the Power Indicator light up on the Headphones.

## Multifunction Button

Answer or end a phone or video call

- ▶ If you receive a phone call to a connected mobile device while wearing your Headphones, press the Multifunction Button to answer or hang up.

Pause or play your music

- ▶ If listening to music, press the Multifunction Button to pause or play your music.

## Carry case

Travel with your Headphones

- ▶ Clip the provided Carabiner to the carry case for easy travel.

# Connect your TV Streamer

Follow the instruction overleaf to learn how to:

1. Power your TV Streamer.
2. Connect your TV Streamer to your TV with three options depending on your TV audio output.

## PLEASE NOTE


**Your Audeara Headphones are pre-paired to the TV Streamer out of the box**

To watch TV with your Headphones, simply set up your TV Streamer and turn on your Headphones.

## 1. Power your TV Streamer

The TV Streamer is powered by the USB Power Cable provided

- ▶ Connect the USB-C end (smaller tip) into the back of the TV Streamer.
- ▶ Connect the USB-A end (bigger tip) into your TV **OR** a USB wall charger. Note: USB wall charger is not provided.

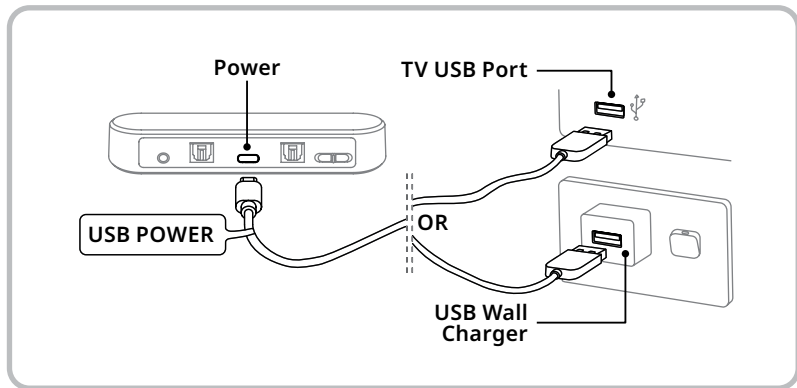When powered on, you will see the TV Streamer home screen

- ▶ If you don't see this screen, ensure your TV or wall charger is powered on.
- ▶ For more information about the home screen, see page 9.

## 2. Connect your TV Streamer to your TV

There are three options depending on your TV audio output.

Choose the option that's right for you and follow the instructions overleaf to connect to your TV.

### OPTION A

- ▶ For TVs with Digital Optical Audio Output
- ▶ Most modern TVs will require this option

### OPTION B

- ▶ For TVs with 3.5mm Headphone Audio Output

### OPTION C

- ▶ For TVs with RCA Audio Output (red/white ports)

### PLEASE NOTE

#### Turning your TV Streamer on/off

There's no need to turn your TV Streamer off. However, if you wish to do so, hold down the middle Multifunction Button for 3 seconds until powered on or off.

## OPTION A


### TV with Digital Optical Audio Output

Connect the TV Streamer to the TV with the provided Digital Optical Audio cable

- ▶ Remove plastic caps on both ends of the cable.
- ▶ Plug one end of the cable into the TV Streamer at Digital In, and the other end into the Digital Optical Output of your TV.


Select Digital as the Audio Input option on the TV Streamer

- ▶ Slide the Audio Input Switch to the right to select Digital.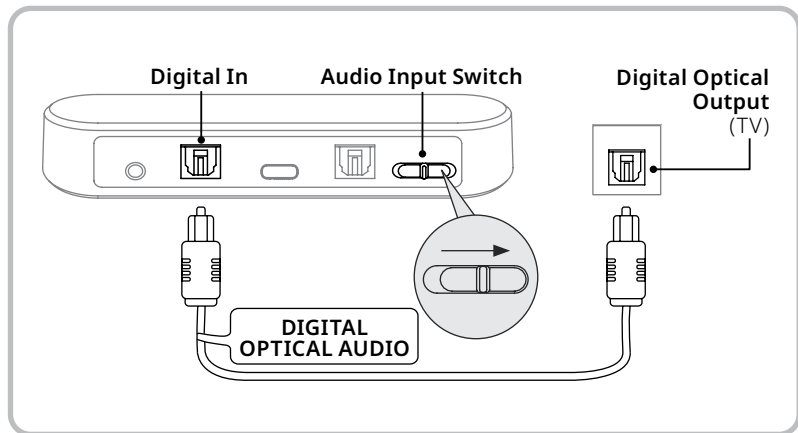

Ensure PCM is selected in your TV audio settings

- ▶ Open the Settings menu on your TV.
- ▶ Select Sound Settings, then select either: Additional Settings / Advanced Settings / SPDIF Settings.\*
- ▶ Select either: Digital Sound Output / Audio Format.^
- ▶ Select PCM.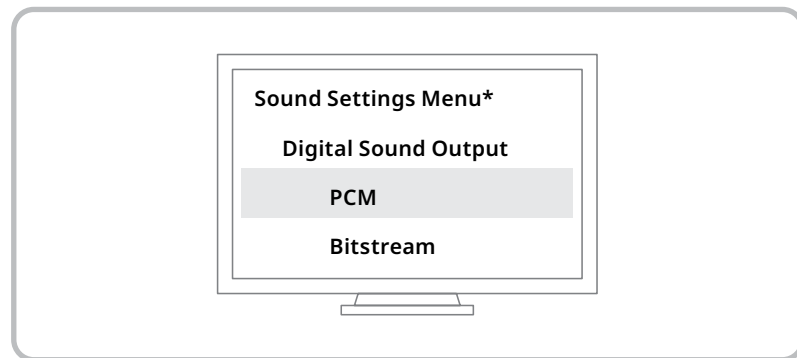

#### PLEASE NOTE

\* Every TV is different – the appearance and language of the settings menu may vary slightly depending your TV.

^ HDMI Audio Format is not the same as Audio Format.

## OPTION B


### TV with 3.5mm Headphone Audio Output

Connect the TV Streamer to the TV with the provided Analog Audio cable

- ▶ Plug one end of the cable into the TV Streamer at Analog In, and the other end into the Headphone Audio Output of your TV.

Select Analog as the Audio Input option on the TV Streamer

- ▶ Slide the Audio Input Switch to the left to select Analog.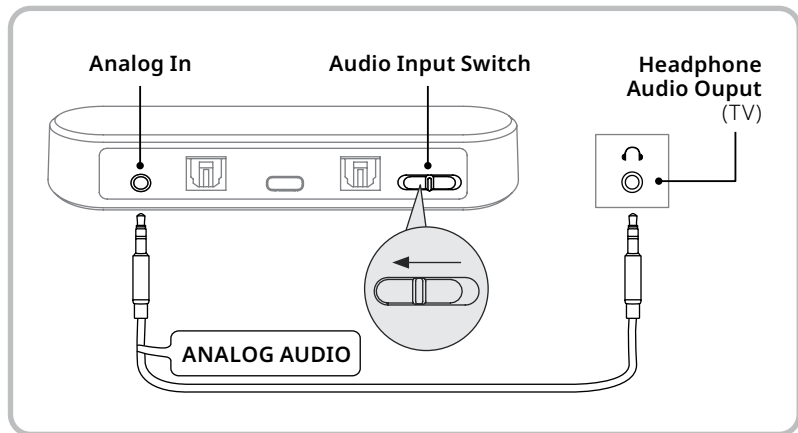

## OPTION C


### TV with RCA Audio Output (red/white ports)

Connect the TV Streamer to the TV with the provided Analog Audio and AV Adapter cables

- ▶ Plug one end of the Analog Audio cable into the TV Streamer at Analog In, and connect the other end to the AV Adapter.
- ▶ Plug the other ends of the AV Adapter into the matching red/white RCA Audio Output ports in your TV.

Select Analog as the Audio Input option on the TV Streamer


- ▶ Slide the Audio Input Switch to the left to select Analog.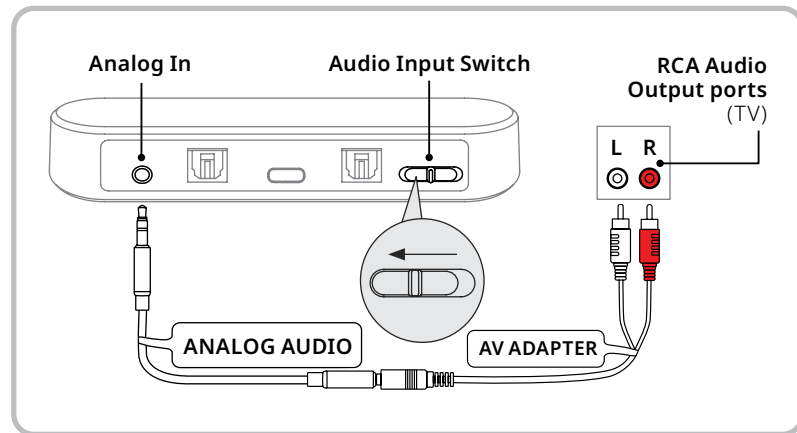

# Additional devices


## Connected device(s)

Headphone icons will have an A or B to differentiate between connected devices.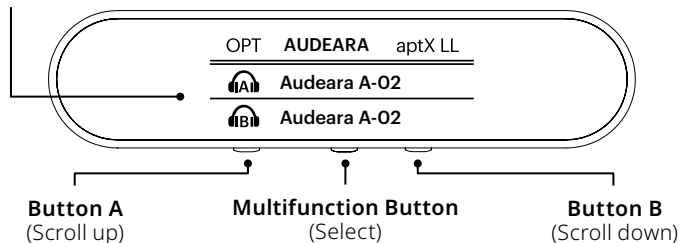

## Connect to a second pair of Headphones

- ▶ Put your Headphones into pairing mode (refer to Step 1 in the 'Connect your Headphones' section of this guide).
- ▶ Hold down button A or B on your TV Streamer for 3 seconds; the screen will now say "Searching..."
- ▶ Using buttons A and B to scroll up and down, navigate to the Headphones you wish to connect to on the screen.
- ▶ Select the Headphones by pressing the Multifunction Button on your TV Streamer. You will hear an audible prompt from your Headphones that says "Connected".

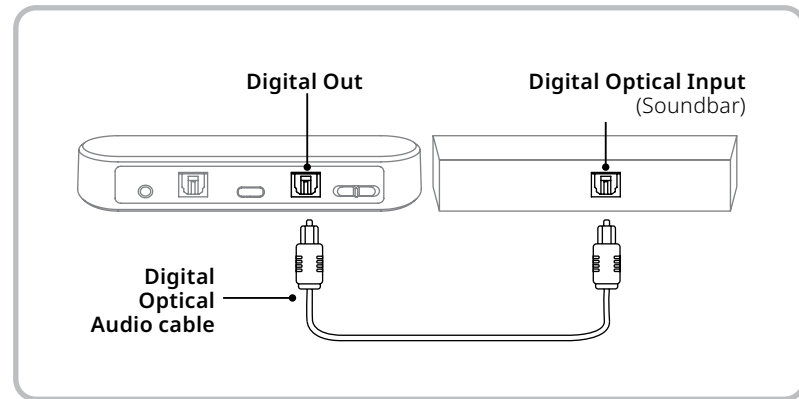## Connect a sound bar

- ▶ This option is only available if you have already connected your TV Streamer to your TV with Option A in the 'Connect your TV Streamer' section of this guide.
- ▶ To connect to your sound bar, you will need an additional Digital Optical Audio cable (not provided). Plug one end into the TV Streamer at Digital Out, and the other end into the soundbar.

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