## **CA-BX FAQ**

- Q: What is the range of the CA-BX?
- A: It can work up to 200ft away from the CA-360
- Q: Where should I place the CA-BX?
- A: For optimal results, place it about 2-5 feet away from your baby but keep it out of reach.
- Q: Can I set the CA-BX such that I will only be alerted when my baby cries louder than usual?
- A: Yes, There is a visual loudness indicator to guide you for proper sensitivity setting. See your user manual for more details.
- Q: Where should I place the CA-BX?
- A: For optimal results, place it about 2-5 feet away from your baby but keep it out of reach.
- Q: Can I set the CA-BX such that I will only be alerted when my baby cries louder than usual?
- A: Yes, There is a visual loudness indicator to guide you for proper sensitivity setting. See your user manual for more details
- Q: How many CA-BX can I have with my CA-360?
- A: You can have up to three CA-BX.
- Q: The CA-BX seems to be working and lights up when pressed, but why is my CA-360 not responding?
- A: Most likely, your CA-BX is not connected with your CA-360 yet. Make sure that your CA-360 is paired with your CA-BX. Refer to your CA-360 for the pairing instructions. Another possibility is that your CA-BX is out of range, over 200 ft away from your CA-360. You can fix this problem by simply bringing them closer together.
- Q: How does the CA-BX monitors my baby's crying?
- A: It uses a microphone to pick up your baby's crying sound.