

# CA-MX FAQ

**Q: How many CA-MX can I install with my CA360?**

**A:** You can use up to 2 CA-MX with each CA360

**Q: How far away from the CA360 can I place the CA-MX?**

**A:** You can place the CA-MX up to 200ft away from the CA360

**Q: The CA-MX seems to be working and lights up when pressed, but why is my CA-360 not responding?**

**A:** Most likely, your CA-MX is not connected with your CA-360 yet. Make sure that your CA-360 is paired with your CA-MX. Refer to your CA-360 for the pairing instructions. Another possibility is that your CA-MX is out of range, over 200 ft away from your CA-360. You can fix this problem by simply bringing them closer together.

**Q: Can I use the CA-360 with the motion sensor of my existing alarm system?**

**A:** No