



DIRECTIONS FOR REACTIVATING DESICCANT:

1. Before retiring for the night, clean your hearing aids (and ear molds if you have them) with AudioWipes™ cleaning wipes
2. Open battery doors
3. Place the hearing aids on the screen side of the desiccant
4. Replace the lid and close tightly

NOTE:

When all the desiccant beads are green, it is time to reactivate. Remove desiccant from the jar and reactivate desiccant in the microwave or the oven

TO REACTIVATE IN MICROWAVE

Place desiccant, screen side up, in microwave and heat for 30 seconds on high. Allow to cool for 20 minutes.

Most beads will have a yellowish tint which means the desiccant is ready to use. If not, heat for another 30 seconds

TO REACTIVATE IN CONVENTIONAL OVEN

In a conventional oven, remove desiccant from jar and back, screen side up, for one (1) hour at 200° (93°C), or until beads are yellow. Allow desiccant to cool before handling