Hear me out: EAR DRYER MD™ helps dry your ears of excess water with the push of a button. Three cheers for dry ears!

With this easy-to-hold, portable device drying your ears after swimming or showering has never been easier. Your ears will be as dry as our humor.

## DIRECTIONS FOR USE:

- Fully charge device and unplug before use. The lights on the front of the device indicate it's charging and the light stops blinking when fully charged.
- Remove the tip cover and press and hold the power button on the right side of the device until you hear the beep to turn it on. Note: it is recommended to preheat the device in drying mode for about a minute to increase efficacy.
- Carefully place the end of the drying tip near the external ear canal. Do not push the tip too far into the ear canal to avoid injury. Dry excess water in the ear canal. Repeat on other ear if necessary.
- Press and hold the power button until you hear the beep to turn off the device.
  Remove tip and wash the tip with soapy water or with an alcohol towelette after each use.

## WARNINGS:

## This product is not recommended for use for children under 12 unless recommended by your healthcare provider.

- Consult healthcare provider to answer personal questions before use.
- . Do not use this product if you have ear pain, ear drainage, ear rash, or ear tubes.
- Do not aggressively shove or stick this product into your ear canal as it may cause bleeding, discomfort, or damage to the eardrum.
- Use only in a stationary environment. If using on children, hold child's head firmly to avoid jumping or movement.
- Discontinue use and immediately turn device off if device becomes hot or if warm air becomes hot.
- Disinfect the tip attachment after each use with an alcohol towelette. Tip is not dishwasher safe.
- . Not for use in nostrils or anywhere else in your body
- . Never use this device while bathing or near water. Do not submerge in water.