EAR WAX MD[®]

Clinically Proven

Hey you, put down the cotton swab.

Although it's tempting to use, cotton swabs can push wax farther into the ear canal, possibly causing earwax impaction or injury. With Earwax MD[®], those ear-clogged days are over.

How do I know if my ear is clean?

After using Earwax MD, your ears may feel clearer or cleaner. Another way to know if you ear is clean is to have a medical professional examine your ear with an otoscope.

What do I do if EARWAX MD doesn't work for me?

If Earwax MD does not work after one to two 15-minute treatments, repeat treatment and rinse thoroughly. If desired results are not reached, you may need to consult your doctor for a professional cleaning. Following a professional cleaning, use Earwax MD at home for routine maintenance. Ask your doctor how often you should use Earwax MD (e.g.: twice per month, once per month, etc.).

Clinical Data

The proof is in the data. In a clinical study, 86% of ears tested with Earwax MD had a complete clearance of earwax in one to two 15-minute treatments. In a separate study, Earwax MD was shown to be non-irritating to the skin.

Warnings:

- For external use only.
- Discontinue use if you experience discomfort using this product.
- Avoid contact with the eyes.
- Only use warm water to rinse. Using cold water or hot water can cause dizziness, vertigo, or nausea.
- Do not use if you have a perforation (hole) or injury in the eardrum or if you have tubes in your ears.
- If swallowed, immediately call Poison Control 800-222-1222
- For children under 2, consult doctor before use.

Customer Service

For the most efficient customer service, call us at 844-732-7929 or email hello@eosera.com.

Stay connected!

Visit www.eosera.com to learn more about our line of ear care product and to view instructional videos. Whether it's for earwax removal, ear pain, ear itch, or routine ear cleaning, Eosera has what you need.

FOLLOW @EOSERAINC ON SOCIAL MEDIA

HOW TO USE EARWAX MD IN 6 EASY STEPS

Step 1

Lie down on your side in a comfortable position. Be sure your ear is parallel to the surface so that nothing drips out of your ear. Place a towel on your shoulder just in case anything drips.

Note: Remove hearing devices before use.

Step 2

Take the bottle of Earwax MD and fill the dropper with the product.

Step 3

With your head tilted to the side, slowly fill the ear canal with Earwax MD. All ear canals are shaped differently, so the number of drops required to fill the ear canal will vary. Once your ear canal feels full, you are finished with the drops.

Step 4

Wait for 15 minutes for Earwax MD to work.

Step 5

After 15 minutes, it's time to rinse your ear. There are several ways to rinse your ear:

- Fill the purple bulb (only included in the Earwax MD Kit) with warm water and lean your head over the bathroom sink to flush the ear canal. Using water that is too hot or too cold may result in temporary nausea, vertigo, dizziness, or discomfort.
- For the easiest and most effective rinsing, use Eosera's Wax Blaster MD[®] (sold separately) to flush the ear canal with a warm water rinse.
- Stand under the shower with your head tilted to let warm water stream into your ear canal and flush out the wax.

If earwax does not dissolve in 15 minutes, repeat treatment.

You may see cloudy water or wax coming out in chunks. This means it's working! Depending on the amount of wax in your ear canal, you may need to rinse several times to fully flush out the wax. If you do not see chunks of wax, it could be because the formula effectively dissolved the wax. If you feel your ears are clearer after following our directions for use, Earwax MD has done its job.

Step 6

Repeat steps 1-5 on your other ear.

To view a how-to video scan the QR code

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