## EARDOC MASSAGER 4055

# Relieves ear pain

The Eardoc™ is a safe, simple, efficient and non-invasive device that solves the problem, rather than treats the symptoms.

The Eardoc™ is a device designed to open the ear tube and allow fluid and air to drain, for adults and for children. Divers and air travelers that suffer from the pressure know this accumulation of fluids and air too well. The Eardoc™ helps drain fluids and air, lessening the pressure on the eardrum, allowing instant relief of the ear pain - no pills needed.


How does Eardoc™ work: Eardoc™ generates and transmits vibration waves through the mastoid bone to the Eustachian tube. This allows the Eustachian tube to open in a non-invasive drug free way. The waves opening the Eustachian tube allow drainage of the trapped fluids and air, and ease the pressure.

To view a video about how the Eardoc™ works go to: http://www.eardoc.info/ Or scan this code:



#### The Eardoc™ is a safe and simple treatment for:

Otitis Media Trapped fluids and air in the middle ear Eustachian tube dysfunction Air travelers with ear pressure during landing/Take-off. Divers suffering from excessive pressure in the ear

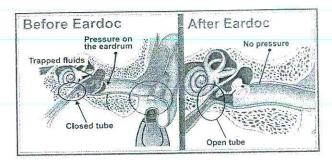

#### How to use the Eardoc™:

Insert 2 AA batteries according to signs (+) and (-) on the batteries and tray. Insert batterys in

- to the Eardoc™ battery ray and close the lid.

  1. Turn on Eardoc™ by pressing the upper button. This will turn on the Eardoc™ and the blue light on the Eardoc™.
- By pressing the upper button, the strength of the vibrations will increase. The Eardoc™ has 6 available speeds.
- Place the Eardoc™ behind the ear (see diagram on the right).
- Set the speed to a comfortable strength using the buttons.
- After use, turn the Eardoc™ off by pressing the lower button until it stops and the light turns off



Want to see a video? Scan this QR code in your phone -





The Eardoc™ research: the Eardoc™ was studied in a European university. The study checked children with an audiometer - a device that checks the fluids in the middle ear. The study shows significant reduction of


middle ear fluids.

All the childre in the research had Ear infections and/or fluids accumulated in the middle ear. Group A are children with ear infections but without fluids. Group B are kids with middle ear infections and/or fluids treated by Eardoc<sup>TM</sup>. As seen on the left, the number of healthy children in group A decreased while in group B the healthy children number increased significantly while using the Eardoc<sup>TM</sup>.

### Eardoc users are extremely satisfied!

The Eardoc™ helped 94 percent of the purchasing customers (from a 600 customers survey!). 98 percent of the Eardoc™ users will use it again to prevent and handle the Ear infection

Temporary relief of pain

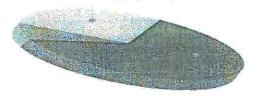

Diel recht beetpe at sall

## The pain eliminated completely

The pain eliminated completely
 Temporary relief of pain
 Did not help at all
 The pain is even stronger now

60.38% 33.96% 5,66% 0%

#### Eardoc™ Warranty and Service:

Manufacturer warrants the EARDOC (the "product") against defects in materials and workmanship under normal use for a period of 6 month (six) from the date of purchase by the original purchaser & display of purchase receipt ("warranty period"). Under this limited warranty, if a defect arises and a valid claim is received by Manufacturer within the warranty period, at its option and to the extent permitted by law, Manufacturer Will either (1) repair the product at no charge, or (2) exchange the product for a new or refurbished product. In the event of a defect, to the extent of the permitted law, these are your sole and exclusive remedies. A replacement product or part assumes the remainingwarranty of the original product or ninety (90) days from the date of replacement or repair. Any replacement after fourteen days from the date of purchase will be charged 17 (seventeen) USD handling charges and shipment cost. When a Product or part of the Product is exchanged as part of warranty service, any replacement item becomes customer property and replaced item becomes Manufacturer's property. Parts provided by Manufacturer must be used in Products for which warranty service is claimed. When a Product receives warranty service, Manufacturer at its option of repairing or replacing the defect. Manufacturer reserves the right to subcontract the product warranty services to be provided hereunder to qualified third parties.

Obtaining Warranty Service:

Please contact Customer Support at www.EARDOC.info for specific instructions about how to obtain warranty service for your device. In general, you will need to deliver your Product, including all components, postage and handling charges paid, in either its original packaging or in equally protective packaging to the address specified by Customer Service. Please note international shipping may be needed to our lab location. Before you deliver the Product for warranty service, it is buyer's responsibility to verify with Kencap Ltd. and receive a written approval number, and confirmation of the need for shipping and servicing.

#### Limitations:

This Limited Warranty applies to the product only. This Limited Warranty does not apply to a Product or part of the Product, plastics that are physical broken, not used according to instructions and/or has been altered or modified by anyone who is not a representative of Manufacturer. In addition, this Limited Warranty does not apply to damage caused by: (a) use with non-original products; (b) direct or indirect damage including third party (c) operating the Product outside the permitted or intended uses described by manufacturer, (d) improper voltage or power supply; or (e) service (including upgrades and expansions) performed by anyone who is not a representative of the manufacturer. This Limited Warranty is not transferable.

TO THE EXTENT PERMITTED BY LAW, THE WARRANTY AND REMEDIES SET FORTH ABOVE ARE EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES AND REMEDIES, ORAL OR WRITTEN, EXPRESS, IMPLIED OR STATUTORY. MANUFACTURER MAKES NO OTHER REPRESENTATIONS OR WARRANTIES, INCLUDING, BUT NOT LIMITED TO, WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND WARRANTIES AGAINST HIDDEN OR LATENT DEFECTS. IF WE CANNOT LAWFULLY DISCLAIM STATUTORY OR IMPLIED WARRANTIES, THEN TO THE EXTENT PERMITTED BY LAW, ALL SUCH WARRANTIES SHALL BE LIMITED IN DURATION TO THE DURATION OF THIS EXPRESS LIMITED WARRANTY AND TO REPAIR OR REPLACEMENT SERVICE AS DETERMINED BY US IN OUR SOLE DISCRETION. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. WE ARE NOT RESPONSIBLE FOR DIRECT, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM ANY BREACH OF WARRANTY OR UNDER ANY OTHER LEGAL THEORY. SOME STATES DO NOT ALLOW THEEXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE EXCLUSIONOR LIMITATION MAY NOT APPLY TO YOU.

This limited warranty gives you specific legal rights, and you may have other rights which vary from state (country) to state. For Service inquiries, please contact Customer Support

This limited warranty gives you specific legal rights, and you may have other rights which vary from state to state. For Service inquiries, please contact Customer Support.

Copyright 2011 Kencap Ltd. all Rights Reserved. Kencap, Eardoc™, Eardoc.info, Earduc, and the Eardoc logo, DOC-line, are trademarks of Kencap Ltd

#### Important safety information

#### Warning

- Use Eardoc™ only for treating earaches and only according to instructions. Children under 6 should use the Eardoc™ with adult supervision.
- Do not use in water
  - Do not use for children under 1 year old.
- Eardoc™ does not replace medical treatment or medication.
- If pain persists see a doctor.
- Recommended to use while sitting.

Want to let us know your thoughts on the Eardoc?

You can contact us via: E-mail: info@eardoc.info

Twitter: @eardoc

Facebook: Eardoc

### Other Products from DOC line




All this product and more, are patented in several countries and Copyrighted. All Rights Reserved. DOC, EARDOC, KENCAP, URIDOC, HEADOC and the GUMDOC logo, are trademarks and belongs to the inventor and/or to Kencap Ltd.

Want to let us know your thoughts on the Eardoc?

You can contact us via: E-mail: info@eardoc.info Twitter: @eardoc

Facebook: Eardoc

## Other Products from DOC line

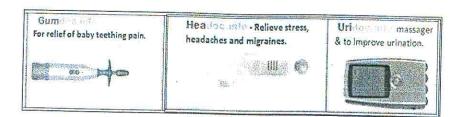

All this product and more, are patented in several countries and Copyrighted. All Rights Reserved. DOC, EARDOC, KENCAP, URIDOC, HEADOC and the GUMDOC logo, are trademarks and belongs to the inventor and/or to Kencap Ltd.