ERGO INSTRUCTION MANUAL



WIRELESS REMOTE BUTTON



Recharge Remote 4 Hours before using (Use receiver power USB cable)
LED –Yellow when charging
LED-Green when fully charged

PATIENT INSTRUCTIONS:

Press blue dot to respond with finger or thumb from either hand

The remote will flash the LED red and gently vibrate confirming their response.

CLEANING -

Wipe down remote after each use with an antibacterial wipes (DO NOT USE ALCOHOL ON UNIT)


TO CHANGE CHANNELS

Reposition DIP switches to be of the same pattern on the remote and receiver.




Default is All "ON"

PEHRATEK PRODUCTS 9285 Pierson Lake Drive Chaska, MN 55318 (952) 470-9063

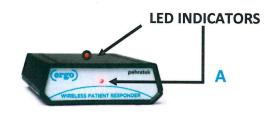

WIRELESS SIGNALLING RECEIVER

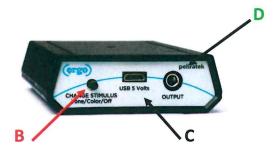

A-Position near audiometer for best visibility and or audibility to the examiner

C Plug in USB power adapter to power up unit

B Select desired signaling mode by pressing button for 2 seconds to cycle through options. Press again to test selection.

Red LED-No sound Red Flashing-No sound Green LED-No sound Red LED –Low sound Red LED-High sound

D OPTIONAL-Connect output to audiometer to utilize built in patient response light (adapter included)

WHITE	Standby
BLUE	ACTIVATED
RED FLASHING	LOW BATT
YELLOW	CHARGING
GREEN	FULL CHARGE