

REORDER NO. H59200



SANI-CLOTH® AF3 GERMICIDAL DISPOSABLE WIPE

Bactericidal, Tuberculocidal, Virucidal** in 3 minutes
Fragrance Free

ACTIVE INGREDIENTS:

n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl
ethylbenzyl ammonium chlorides.....0.14%
n-Alkyl (60% C₁₄, 30% C₁₆, 5% C₁₂, 5% C₁₈)
dimethyl benzyl ammonium
chlorides.....0.14%

OTHER INGREDIENTS.....99.72%
TOTAL.....100.00%

KEEP OUT OF REACH OF CHILDREN

CAUTION: See back panel for additional
precautionary statements

NET CONTENTS: 1 LARGE WIPE 5 x 8 in (12.7 x 20.3 cm)

Net Wt. 0.17 oz. (4.9 g)

EPA REG. NO.: 9480-9

EPA EST. NO.: A-9480-NY-1, C-72956-AR-1

Alpha character will precede batch code on product

Area of Use:

Hospital, Healthcare, and Critical Care Use Sites:
May be used on hard nonporous surfaces of: blood
glucose meters; patient support and delivery
equipment. This product is not to be used as a terminal
sterilant/high level disinfectant on any surface or
instrument that (1) is introduced directly into the
human body, either into or in contact with the
bloodstream or normally sterile areas of the body, or
(2) contacts intact mucous membranes but which does
not ordinarily penetrate the blood barrier or otherwise
enter normally sterile areas of the body. This product
may be used to preclean or decontaminate critical or
semi-critical medical devices prior to sterilization or high
level disinfection.

Directions for use: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **CLEANING AND DEODORIZING:** Wipe surface with towelette until clean. Let air dry. For disinfecting hard nonporous surfaces only. **TO DISINFECT:** Use a wipe to remove heavy soil. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous 3 minute wet contact time. **PRECAUTIONARY STATEMENTS:** Hazards to humans and domestic animals **CAUTION:** Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using toilet. **FIRST AID If in eye:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment. **STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal. **Storage:** Store in dry location. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Packet Disposal:** Put empty packet in trash collection.

**NOT A SKIN WIPE
FOR USE ON HARD
SURFACES ONLY**

Manufactured by: Professional Disposables International, Inc.
Orangeburg, NY 10962-1376 • USA
1-800-999-6423



2AH592