

To activate the rechargeable battery

1. Slide open the battery compartment on the bottom of the HA360M-II by pushing left on the arrow and lifting on the opposite end and the same time (see link below for video).
2. Take the battery out and plug the connection wire into the top right corner of the battery compartment. Replace the battery and slide the battery door back into position (see link below for video).

<https://www.youtube.com/shorts/OPlulqpW7-s> - (Link to HomeAware-II Opening Battery Door)

<https://www.youtube.com/shorts/SiLs69OzULk> - (Link to HomeAware-II Battery Connection)

HA360M-II power supply

1. Plug in the power supply in the "5.1V 2A DC IN" port on the back of the unit. There is a cover over the "5.1V 2A DC IN" port, push the cover down until it snaps off and plug in the power supply. Once the power supply is inserted snap cover back into place pushing up. Plug the other end of the power supply in a 120V outlet (see the link below for video).

<https://www.youtube.com/shorts/wvml1dBHm5U> - (Link to HomeAware-II Plug Cover)

Message that may scroll across the display before setup is completed

1. MISSING VIBRAT will scroll across the display until the vibrator is plugged in if you are using the additional accessory option (HA360V-II) vibrator. If you are not using the vibrator, your selection should be set to OFF so that the alert does not trigger needlessly. It comes from the factory set to ON.
2. Press SET UP button and the word LINK will be shown on the display.
3. Press the TONE button until the word V SENS is shown on the display, press ENTER.
4. Press the TONE button to toggle between ON and OFF, press ENTER on your selection.

Setting the Time of Day

1. Press the TIME SET button. Blinking red dots shown on the display indicate time set mode has been activated.
2. Press the HOUR buttons up or down and the MINUTES buttons up or down to set desired time. The PM indicator is located on the top left-hand side.
3. Press the ENTER button when done to complete time setting. Display dots will stop blinking to indicate time has been set.

Setting Alarm Times - there are two alarms AL1 and AL2.

Setting Alarm 1

1. Press and hold ALARM 1 SET button. Blinking red dots shown on the display indicate alarm set mode has been activated.
2. Press the HOUR buttons up or down and the MINUTE buttons to set desired alarm time.
3. Press the ENTER button when done to complete, a red LED light will appear on the display's left side indicating AL1 is now set.

Setting Alarm 2

Follow the same steps for alarm 1 above to set alarm 2 it will have its own indicator light also located on the left as AL2.

Turning Off Alarm

To turn off alarm AL1 or AL2 press ALARM OFF or SNOOZE button. If you want to turn off the alarm completely, press ALARM 1 SET or ALARM 2 SET. The red indicator light will go off depending on the alarm you are shutting off.

Setting Alarm Duration

Alarm duration will allow you to select the length of time that the alarm will run (5 min to 30 min).

1. Press SET UP button and the word LINK will be shown on the display.
2. Press the TONE button and the word CLOCK will be shown on the display, press ENTER, the word ONTIME will be shown on the display, press ENTER.
3. Press the TONE button to set the desired alarm duration (5 min – 30 min) and press ENTER to complete alarm duration setting.

Setting Snooze Duration

Snooze duration will allow you to select the length of time that the snooze alarm will run (5 min to 30 min).

1. Press SET UP button and the word LINK will be shown on the display.
2. Press the TONE button and the word CLOCK will be shown on the display, press ENTER, the word ONTIME will be shown on the display.
3. Press the TONE button until the word SNOOZE is displayed, press ENTER. The default snooze duration is set for 10 minutes if this is your desired snooze duration press ENTER. If you would like a shorter or longer snooze duration (5 min to 30 min) press the TONE button to toggle until desired snooze duration is displayed and press ENTER.

Automatic Display Dimmer

The HA360-II has a built in auto dimmer. It will automatically dim when the room gets darker. The auto dimmer can also be turned off and will stay bright even when the room gets dark.

Turning Off the Automatic Dimmer

1. Press Set Up button and the word LINK will be shown on the display.
2. Press the Tone button and the word CLOCK will be shown on the display.
3. Press the Tone button until the word DIM is shown on the display, press Enter.
4. Press the Tone button to toggle between ON or OFF, press Enter on your selection.

Speaker Volume and Tone Setting

1. Press the VOLUME up and the HOUR down buttons to select the preferred volume level. There are 8 volume levels and an OFF level to completely turn off the sound.
2. Press the TONE up and the MINUTE down buttons to select the preferred tone level. There are 15 different tones to select from.

Alerts Option (Number of Alerts)

The HA360M-II has alert options that will let the user pick how many times they will be alerted 3, 6 or 9 times.

Setting the Alerts Option

1. Press SET UP button and the word LINK will be shown on the display.
2. Press the TONE button until the word ALERTS is shown on the display, press ENTER.
3. Press the TONE button to toggle between 3, 6, or 9, press ENTER on your selection.

Smoke / CO Alert - Detecting the Different Alert Sounds

- The Main Unit detects loud sounds and will alert you to the source of that sound. If it detects a loud sound from a smoke detector that meets the T3 standard used for SMOKE DETECTORS, "SMOKE" will scroll on the display of the HA360M-II.
- If it detects a loud sound from a CARBON MONOXIDE (CO) DETECTOR that meets the T4 standard, "CO" will scroll on the displayed of the HA360M-II.