

Aqua Plus Sealant, aka Abdrucklack

JKR10-SEAL

Instructions for Use

Prior to applying lacquer (dipping or brush on) the impression must be absolutely clean, dry and free of grease. Use an appropriate surface cleaner (e.g. isopropyl alcohol). After shaking off excess lacquer, let the impression air dry at room temperature for approximately 15 minutes. The surface of the impression should now show a glossy layer

IMPORTANT: close container carefully after each use

NOTE: during application of larger amounts of lacquer, use a suction system. Please refer to safety data sheet