
Jodi®- Pro

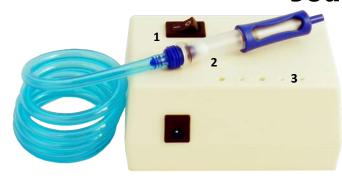

- 1. Vacuum On/Off Switch
- Vacuum Filter with ball, please don't clean, they can be filled up and then replaced inexpensively
- 3. Inserts for Needle storage and reamers.

NORMAL VACUUM OPERATION

- 1. Turn on switch (#1) to Vacuum.
- 2. The ball will rise towards the connector when air is flowing, it will fall towards needle when the needle is plugged.
 - The ball will also drop when it bottoms out against the receiver or the filter is plugged.
- 3. Our pink needle is specifically designed for maximum vacuum efficiency in CIC's, ITC, ITE, and RIC sound bores.

The Light Blue needle is sized for open ear Thin Tubes. Our Black needle is oversized for Ear Molds. Use of force will bend these needles and ruin Hearing instruments.

Changing these needles to another size will decrease vacuuming efficiency in the sound bore and the receiver nubbin.


CLEANING

DO NOT IMMERSE IN LIQUIDS OR APPLY LIQUIDS BY POURING ONTO UNIT. FIRST—Test your preferred cleaning product on a small area.

Alcohol or household ammonia is very effective.



- 1- Spare Filter
- 1- Blue Needle and Reamer
- 1- Reamer for Pink and Black
- 1- Pink Needle
- 1- Black Needle

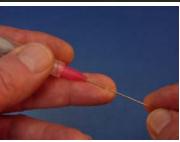

USING REAMER Create an indent on your finger with needle, place reamer in the indent and slide into needle.



NOTES

A plugged needle will cause flow indicator ball NOT to rise.
A plugged needle will act as a ramrod and push wax deeper.
Pushing needle in too fast without paying attention will produce undesired results.
Pushing too hard on Microphone will dislodge Mic. from faceplate
Old Receiver tubes are prone to coming loose, be gentle.

READ ALL INSTRUCTIONS BEFORE USING IMPORTANT SAFEGUARDS

THIS PRODUCT IS INTENDED FOR THE CLEANING OF HEARING AID DEVICES AND NOT TO BE USED TO CLEAN EARS.

DANGER – to reduce the risk of electric shock:

- 1. Always unplug this product immediately after using.
- 2. Do not use while bathing.
- 3. Do not place or store product where it can fall or be pulled into a tub or sink.
- 4. Do not place in or drop into water or other liquid.

WARNING – To reduce the risk of burns, electrocution, fire or injury to persons:

- 1. Do not get the syringe end near eyes or ears, as the needle can puncture skin.
- 2. Do not clean ears with this product.
- 3. If your product comes with a Desiccant, and you are drying it out by use of a heating source—understand that the desiccant will get very hot and will burn your skin.
- 4. This product is intended to clean hearing aids, any unauthorized use will void any and all warranties.
- 5. A product should never be left unattended when plugged in.
- 6. Close supervision is necessary when this product is used by, on, or near children or invalids.
- 7. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- 8. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been damaged, dropped into water. Return the product to the manufacturer for examination and repair.
- 9. Keep the cord away from heated surfaces.
- 10. Never use while sleepy or drowsy.
- 11. Do not use where aerosol (spray) products are being used or where oxygen is being administered.

CAUTION — RISK OF ELECTRIC SHOCK, DO NOT REMOVE COVER. REFER SERVICING TO QUALIFIED PERSONNEL

This product is for use on a nominal 12VDC circuit. Use the appropriate AC to DC adapter as supplied with this product when plugging in to household current. This product is factory equipped with a specific electric cord and plug to permit connection to a proper electric circuit. Make sure that the product is connected to an outlet having the same configuration as the plug. Do not modify the plug provided – if it will not fit the outlet, have the proper outlet installed by a qualified electrician. If the product must be reconnected for use on a different type of electrical circuit, the reconnection should be made by qualified service personnel.

WARRANTY

If wax or debris is found inside unit, it will be assumed that vacuuming occurred without the protection of the supplied filter assemblies. Please do not clean filter assemblies. Replacement filters are available. A \$30.00 cleaning charge may be applied.

Manufacturer will repair or replace the product for all defects and workmanship for a period of 3 Years from the date of purchase.

Syringe assembly and needle are excluded from this or any other warranty.

Manufacturer is not responsible for incidental or consequential damages arising from accident, abuse, misuse, or failure to follow instructions for the product. Any alteration of the product voids this warranty.

To activate this warranty, purchaser must show proof of purchase date of the product. If purchased through JodiVac, your warranty will be automatically activated.

Purchaser is responsible for shipping and handling costs when returning the product for warranty service.