



HOW TO CHANGE OUT OS-03100LED OTOSCOPE BULB

STEP ONE: Twist otoscope head clockwise to remove from otoscope handle



STEP TWO: Bulb located in the bottom portion of otoscope head (see arrow)



STEP THREE: Pull bulb out in direction of arrow below by pinching exposed portion of bulb with thumb and index finger (use or tweezers or similar tool)



STEP FOUR: Insert new bulb in chamber in direction of arrow as shown below



www.oaktreeproducts.com

800.347.1960

otp@oaktreeproducts.com