6 Cleaning

6.1 Cleaning Sentiero

Before cleaning **Sentiero** the device must be turned off and removed from all connected devices.

Make sure that no liquids get to the interior of the device. Do not dunk the device into any liquid, e.g., water or cleaning agents.

Only use wiping disinfection cleaners (Ethanol: 70-80%, Propanol: 70-80%, or Aldehyde: 2-4%). Use a moistened cloth. Please adhere to the local regulations and laws.

6.2 Cleaning of ear probe, headphone, accessory

Please follow the instructions of the manufacturer (see special data sheet of accessory).



Fig. 24: Test coupler (right), probe with mounted probe tip, ear tips (accessories)

6.3 Cleaning and disinfection of the ear probe calibration cavity

The calibration cavity (see Fig. 24) for the ear probe test must be used with a desinfected and clean new probe tip. In case of contamination with pathological material or suspected dirt inside the cavity, do not reuse the calibration cavity. Do not use cleaning liquors or vapour sterilization! For external cleaning, please use a sterile alcohol wipe, typically containing isopropyl alcohol 70%. Please refer to the manufacturer's data sheet for the minimum time period in which the wipe has to be in direct contact with the surface to ensure the effectiveness of the cleaning.