
ReSound Phone Clip+

- Small, lightweight device that clips easily onto clothes
- Sends Bluetooth® phone conversations directly to your patients' hearing aids
- Offers both easy-to-use basic phone operation and state-of-the-art headset functionality
- Doubles as a basic hearing aid remote control
- Onboard hearing aid mute functionality
- High quality stereo sound streaming (A2DP)
- 6 hours talk time, 80 hours standby time

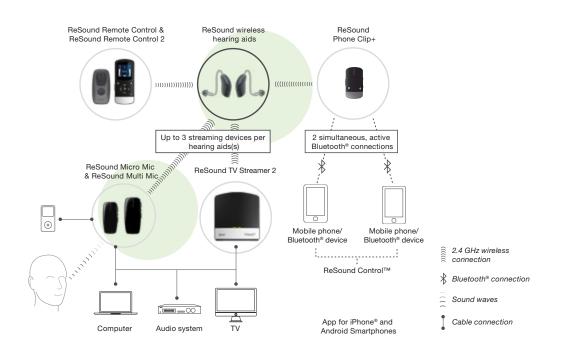

Technical data		
Dimensions	2.7 x 1.26 x 0.5 in	
Weight	0.88 oz	
Power supply/charging	Built-in rechargeable battery	
Battery life	6 hours talk time, 80 hours standby time	
Audio interface s	Bluetooth® phone	
Bluetooth streaming range	Mobile phone to phone clip: over 30 ft	
Bluetooth compliance	Version 2.1	
Bluetooth profile support	Hands Free Profile (HFP), Headset Profile (HSP)	
Wireless streaming frequency	2.4 GHz	
Ear streaming range	Phone clip to hearing aid: 3 ft	
Audio streaming latency	Bluetooth standard: < 40 ms	
Audio quality	Mono 20 kHz sample frequency	
Audio bandwidth	A2DP stereo audio streaming: 100 Hz-10 kHz Phone: 200 Hz-3.4 kHz	
Operating temperature	32° to 113° F	



Quick guide

Before you can use your ReSound Phone Clip+ it is necessary to:


- A. Charge your ReSound Phone Clip+
- B. Pair it with your hearing aid(s)
- C. Pair it with your mobile phone
- D. Connect it to your mobile phone

Consult your audiologist to determine which, if any, of these procedures may have already been completed for you.

A CHARGING YOUR RESOUND PHONE CLIP+

1. Plug the connector end of the charging cable into the Phone Clip+.

RESOUND WIRELESS ACCESSORIES

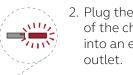

2. Plug the other end of the charging cable into an electric wall outlet.





3. Charging is complete when the

red LED indicator at the top of the

B PAIRING WITH YOUR HEARING AIDS

- 1. Turn your ReSound Phone Clip+ ON by sliding the ON/ OFF switch down to the ON position. A white "I" icon will be visible on the base of the switch and the LED indicator will now blink green every 2.5 seconds.
- 2. Turn off your hearing aids by opening the battery doors on both hearing aids or press and hold the push button for 5 seconds to power down and power up ReSound LiNX Quattro 61 hearing aids.


 Remove the silver cap from the ReSound Phone Clip+.

B PAIRING WITH YOUR HEARING AIDS... continued

4. Press the small white pairing button with the tip of a pen or similar device to put the ReSound Phone Clip+ into pairing mode for 20 seconds.

While in pairing mode the LED will blink yellow every 2 seconds.

 While in pairing mode, turn on the hearing aids by closing the battery doors on both hearing aids.

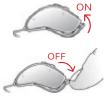

 A melody of beeps will be heard in both hearing aids once they successfully pair with the ReSound Phone Clip+.

Replace the cap on the ReSound Phone Clip+ when pairing is complete.

PAIRING WITH YOUR MOBILE PHONE

- 1. Turn your ReSound Phone Clip+ ON and remove the silver cap.
- 2. Find the connectivity settings on your mobile phone. Look for the "Bluetooth" feature in the main menu or one of the submenus, such as "Connectivity." (Refer to your mobile phone user guide or www.resound.com/Pair-Phone for more detailed instructions.)
- 3. On your mobile phone, ensure that Bluetooth is turned ON.

- 4. Press the small blue Bluetooth pairing button one time to activate Bluetooth pairing mode for two minutes. The LED indicator will turn blue during Bluetooth pairing.
- 5. On the mobile phone search for Bluetooth devices.
- 6. Your mobile phone should present a list of Bluetooth devices discovered. Select "Hearing Aid Phone" from this list.
- 7. Your mobile phone may then ask you to enter a passcode. If so, enter "0000" (four zeros).

Your ReSound Phone Clip+ should now be paired to your mobile phone. Some mobile phones may ask which Bluetooth service you wish to enable. Select "Headset."

D CONNECTING WITH YOUR MOBILE PHONE

- 1. In most instances, the mobile phone will ask you to confirm your intention to connect with the ReSound Phone Clip+ after successfully pairing.
- 2. If you have to manually connect: Locate a list of "Paired Devices" in your phone's Bluetooth" menu. From that list, highlight
 - "Hearing Aid Phone" and select "Connect."
- 3. You are now ready to use your ReSound Phone Clip+.
- 4. On some phones you can configure "Automatic Reconnection." Refer to the user guide of your mobile phone for instructions on how to do this.

Tip: In most instances, a headset symbol on the main screen of your mobile phone will confirm that you have successfully connected.

TIPS AND SUPPORT

For more details about the ReSound Phone Clip+, visit: https://gs.resound.com/en-us/support/support-accessories

108

LIGHT INDICTORS (LED)

The light indicator at the top of your ReSound Phone Clip+ serves as a multipurpose user interface, providing information on the status of your device.

Blinks	Meaning
	Normal operation
	Battery low
	Charging
	Fully charged—still connected to charger
1 1 1	Ready for pairing with hearing aid(s)
	Ready for pairing with Bluetooth device
1 1 1	Bluetooth call active

PHONE OPERATION

In the below table you will find an overview of how to operate the ReSound Phone Clip+ when using it in connection with phone calls.

Note: Except for the answer and end-call operations, any of the operations described below may be phone dependent, that is, they may either differ slightly or be absent altogether.

	1
Answer or end a call	Push the call pick-up/hang-up button once
Reject a call	Double-click the call pick-up/hang-up button
Transfer a call initiated on a mobile phone to the ReSound Phone Clip+	Push the call pick-up/hang-up button once
Put an active call on hold	Press and hold the call pick-up/hang-up button for 2 seconds
Redial the last number	Double-click the call pick-up/hang-up button while a call is neither active nor pending
Use voice dialing	Press and hold the call pick-up/hang-up button for 2 seconds

G REMOTE CONTROL OPERATION

When not on the phone, the ReSound Phone Clip+ can be used as a simple remote control for your hearing aids.

Turn hearing aid(s) volume up	Push "+" on the volume button
Turn hearing aid(s) volume down	Push "–" on the volume button
Mute hearing aid microphones	Push 🚇 once
Unmute hearing aid microphones	Push 🚇 once
Toggle through hearing aid programs	Push D one or more times