

The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

CAUTION:

TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBA475ss is warranted against manufacturing defects in materials and workmanship for (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBA475ss without charge for parts and labor. Simply send the SBA475ss (postpaid) and a copy of your sales slip as proof of purchase to:

Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083

This device complies with part 15 of the Federal Communications Commission Rules. Operation is subject to the condition that this device does not cause harmful interference.

Note: The manufacturer is not responsible for any radio or TV interference cause by unauthorized modifications to this equipment. Such modification could void the user's authority to operate the equipment.



Made in China Sonic Alert 1050 East Maple Rd. Troy, MI 48083 www.SonicAlert.com 1-800-566-3210 V/TTY



America's Leader in Visual Alerting Systems

Sonic Boom Alarm Clock & Bed Shaker Model SBA475ss



OPERATION INSTRUCTIONS

IMPORTANT- Please read these instructions carefully before use and retain for future reference. See operating instructions on back.


Warning: To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting to mains make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC 120V 60Hz.


GENERAL DATA

Power: Consumption: 8w, 120v / 60 Hz (Adaptor AC ~9V 750 MA) UL listed transformer (Underwriters Laboratories)

Made in China Sonic Alert 1050 East Maple Rd, Troy, MI 48083 www.SonicAlert.com 1-800-566-3210 V/TTY

INSTRUCTIONS ANALOG ALARM CLOCK WITH BED VIBRATOR MODEL SBA475SS

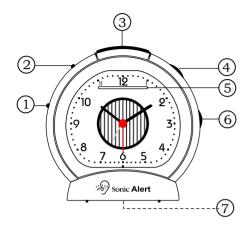

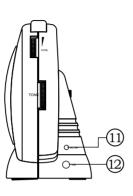

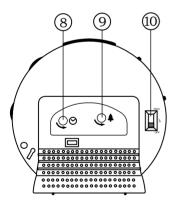




Figure B

Features shown on diagrams above:

- 1) Function Switch
- 2) Back light Hi/Lo Dimmer
- 3) Snooze Button
- 4) Volume Control
- 5) Flashing Light Display
- 6) Tone Control

- 7) Battery Compartment
- 8) Time Set Knob
- o) Time Set Knob
- 9) Alarm Set Knob
- 10) Back light On/Off
- 11) AC Power Input
- 12) Vibrator Input

AC-9V Power Input -

Attach the AC power supply input to the back of clock at AC-9V IN (11) next to the VIBRATOR input, smaller input of the two.

NOTE: IMPORTANT: do not confuse" AC POWER INPUT"(11) with the "VIBRATOR INPUT"(12)

Setting the correct time-

1) Turn the Time Set Knob (8) clockwise until hour and minute hands show desired time.

Setting the alarm time-

1) Turn the Alarm Knob (9) clockwise until alarm hand is at the desired time.

Alarm-

To enable the alarm function, slide the "FUNCTION SWITCH" (1) up to the position that you desire to wake to, you can choose "VIB" for bed shaker, "BUZZ" for sound, and "VIB/BUZZ" for both.

NOTE: whichever function you choose to wake to, will be the same function for both alarms. When the alarm sounds, the Flashing Light Display (4) and the back light will flash until the alarm is stopped.

Snooze Operation-

SNOOZE button (3) - press after alarm has sounded. The alarm will be turned off for 4 minutes for extra sleep. The alarm will come back again after 4 minutes.

Shutting Off Alarm-

Alarm can be shut off by sliding the function switch (1) to off position.

Volume Control-

To adjust the "VOLUME", turn the volume wheel (4) located on right side of the clock.

Tone Control-

To adjust the "TONE", turn the tone wheel (6) located on right side of the clock to the desired tone sound.

Bed Shaker / Vibrator- Figure B

- 1) Attach the bed shaker jack to the side of clock at VIBRATOR input (12).
- 2) To turn on vibrator slide FUNCTION switch (1) to VIB or VIB/BUZZ position.
- 3) Place vibrator under mattress or pillow.

Night Light -

The blue back light can be turned on or off by using switch (10) located at the back of the clock. The brightness of the back light can be adjusted by using the dimmer switch (2) at the side of the clock.

Battery Backup-

To use battery backup, you can install a 1.5-volt AA size alkaline battery at the bottom of the clock located under the battery cover (7). Note the back light, vibrator and sound will not operate during power failure, your alarm and time setting will be saved when power is restored.