

# **INSTRUCTIONS FOR USE**

The Tridal Wave Three Stream Ear Cleaning System is intended to be used for Ear Irrigation

Contents: 1 Tridal Wave Ear Cleaning System



© 2017 Oaktree Products Phone: (800) 347-1960 Web: www.oaktreeproducts.com RM95-0700 Rev. 12/2017

#### **Intended Use:**

The Tridal Wave Three Stream Ear Cleaning System is intended to be used for Ear Irrigation at home.

⚠ WARNING: Only use lukewarm water for irrigation. Using cold water or hot water can cause dizziness, vertigo, or nausea.

Do not use if you have an ear injury, ear infection, perforation (hole) in ear drum, ear drainage or discharge, ear pain, ear tubes, irritation or rash in ear canal or if dizzy.

Discontinue irrigation immediately if bleeding, irritation, or other trauma to the ear canal or tympanic membrane.

## **Assembly Instructions:**

- 1. Separate the sprayer head from the bottle
- 2. Fill bottle with body temperature water.
- 3. After filling, place bottle on secure, stable platform.
- 4. Grasp the sprayer head and guide the tube into the opening of the bottle until fully inserted.
- 5. Secure head with one hand, rotate the fastening ring at the base of the head clockwise until tight.
- 6. Grasp the "blue" Tip and twist onto the luer lock fitting of the adapter wand until it stops.

#### **Use Instructions:**

 Assure that the water temperature is close to body temperature. DO NOT use cold or hot water as it may cause dizziness.

- 2. Prime the unit by discharging into a sink.
- 3. Insert the "blue" tip fully into the ear canal.
- 4. Irrigate as needed while standing over the bathtub or sink. Note, the water will flow freely out of the ear.
- 5. Remove any remaining water in the ear canal with an absorbent sponge.

### **Cleaning Instructions:**

- 1. Prepare a solution of dishwashing liquid and water.
- Ensure bottle and spray head are completely drained of water.
- 3. Wash the outside of the bottle and spray head with the dishwashing solution.
- 4. Fill the bottle with the dishwashing solution and reattach the spray head.
- 5. Shake bottle and pump spray head until soapy water discharges.
- Rinse the outside of the bottle with water until all suds are removed.
- 7. Remove the spray head from the bottle and empty the dishwashing solution from the bottle.
- 8. Rinse and refill the bottle with water, repeat until no suds are visible in the bottle's rinse water.
- 9. Refill the bottle with water and reattach the spray head.
- 10. Pump the spray head until no suds are evident in the discharged rinse water.
- 11. Remove the spray head from the bottle.
- 12. Pump the spray head until all water is discharged.
- 13. Drain all water from the bottle.
- 14. Dry the outer surfaces of the bottle & spray head and leave them separated to allow complete drying.