


ReSound TV Streamer 2

- No need to wear any additional devices besides hearing aids
- Transforms wireless ReSound hearing aids into a set of smart and discreet wireless headphones
- Sends stereo sound from the TV or other audio sources directly to the hearing aids
- Streams crystal clear sound over a distance of over 20 feet—move around while watching TV
- Reconnects automatically if the ReSound TV Streamer 2 user leaves the room and returns within 5 minutes
- With an imperceptible delay of 20 ms, it avoids issues related to synchronization between sound and image



Technical data	
Dimensions	3.25 x 3.5 x 0.8 in
Weight	2.2 oz
Power supply	External wall adaptor, 5 V DC
Power connection	Mini USB
Power consumption	Maximum 150 mA
Audio interfaces	2 x phono line input
Line input sensitivity	750 mv RMS
Wireless connections	Multiple hearing aids per ReSound TV Streamer Up to 3 ReSound TV Streamer 2s per binaural set of hearing aids
Audio streaming range	Over 20 ft
Audio streaming latency	20 ms
Audio quality	Mono/stereo 20 kHz sample frequency
Audio bandwidth	10 kHz
Wireless streaming frequency	2.4 GHz
Emission	Maximum 10 dBm (In compliance with all relevant standards)
Operating temperature	32° to 130° F

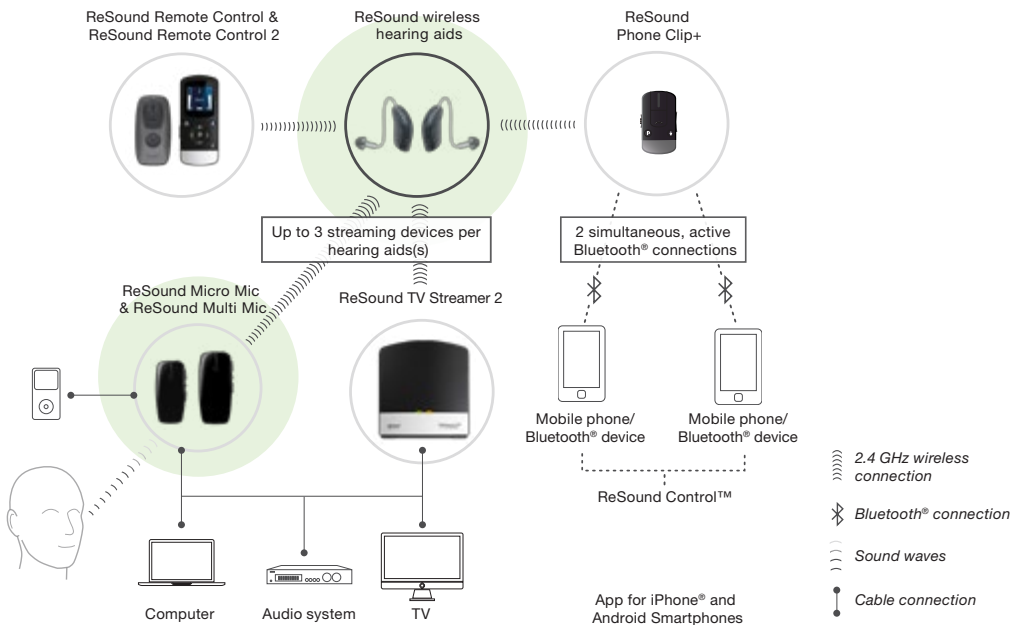

ReSound TV Streamer 2

Quick guide

GETTING STARTED

A PAIRING TO YOUR HEARING AIDS

Note: Check with your hearing care professional to determine if your hearing aids have been pre-paired. Hearing aids that have been pre-paired do not need to be paired again.

1. Make sure the ReSound TV Streamer 2 is plugged into a wall socket and that the green power light is on.
2. Open the battery doors on both hearing aids or press and hold the push button for 5 seconds to power down and power up ReSound LiNX Quattro 61 hearing aids.
3. Press the pairing button on the ReSound TV Streamer 2 once to activate the pairing mode.
4. You can tell that the pairing mode has been activated when channel indicator is flashing. Pairing mode will be active for 20 seconds.
5. While the channel indicator light is flashing, close the battery doors on both hearing aids.

Note: Successful pairing will be indicated by an audible melody played in both hearing aids. You are now ready to use the ReSound TV Streamer 2.



PAIRING TO A SECOND OR THIRD RESOUND TV STREAMER 2

It is possible to pair up to three ReSound TV Streamer 2 units with a given pair of hearing aids.

To pair a second or third ReSound TV Streamer 2:

Follow steps 1 and 2 under **B**

- Then press the pairing button of the second ReSound TV Streamer 2 TWO or THREE times.
- The yellow light indicator on the front of the ReSound TV Streamer 2 will now blink twice or three times every 2 seconds to indicate that it is ready to be paired in channel 2 or 3.

Pairing button pushes	1	2	3	4	5	6
1	1	1	1	1	1	1
2	1 2	1 2	1 2	1 2	1 2	1 2
3	1 2 3	1 2 3	1 2 3	1 2 3	1 2 3	1 2 3

B SETTING UP YOUR RESOUND TV STREAMER 2

The ReSound TV Streamer 2 supports both analog and digital audio input.

Depending on your TV, HiFi stereo, computer or other audio device, the installation of ReSound TV Streamer 2 may be done in several ways described on the following pages.

For **all** installations it is required that you:

1. Plug the power cord into the micro-USB socket on the ReSound TV Streamer 2.
2. Plug the other end of the power cord into an electric wall outlet.

Connect the ReSound TV Streamer 2 to your audio source using one of the following 5 installations.




1. CONNECTING THE RESOUND TV STREAMER 2 TO AN ANALOG AUDIO SOURCE (E.G. TV OR HIFI STEREO)

1. Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the ReSound TV Streamer 2.
2. Plug the other end audio cord into the "AUDIO OUT" terminals on the audio source.




The "AUDIO OUT" is usually marked "L-R" for left and right, and can usually be found on the back of the audio source.

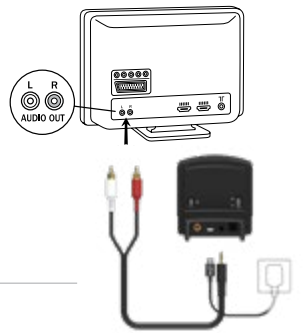


2. CONNECTING THE RESOUND TV STREAMER 2 TO AN ANALOG AUDIO SOURCE USING A HEADPHONE MINI-JACK ADAPTOR

1. Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the ReSound TV Streamer 2.
2. Connect the stereo mini-jack adapter to the red and white plugs at the other end of the audio cable.
3. Plug the stereo mini-jack adapter into the audio source's "HEADPHONE" output terminal.

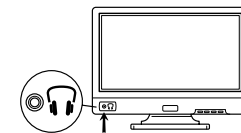

3. CONNECTING THE RESOUND TV STREAMER 2 TO AN ANALOG AUDIO SOURCE USING A SCART ADAPTER

1. Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the ReSound TV Streamer 2.
2. Connect the scart adapter to the red and white plugs at the other end of the audio cable.
3. Plug the scart adapter into the audio source's scart terminal.

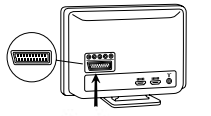


4. CONNECTING THE RESOUND TV STREAMER 2 TO A DIGITAL AUDIO SOURCE USING THE OPTICAL AUDIO INPUT

1. Insert one of the plugs on the optical audio cable into the optical audio input socket on the ReSound TV Streamer 2.
2. Plug the other end of the optical audio cable into the "Digital Audio Out/Optical" on your audio source.




Note: Most optical cables are delivered with a small protective cap to protect the tip of the cable. Please remove this protective cap before inserting the cable.

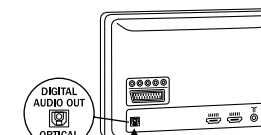

5. CONNECTING THE RESOUND TV STREAMER 2 TO A DIGITAL AUDIO SOURCE USING THE COAXIAL AUDIO INPUT

1. Insert one of the plugs on the coaxial audio cable into the coaxial audio input socket on the ReSound TV Streamer 2.
2. Plug the other end of the coaxial audio cable into the "Digital Audio Out/Coaxial" on your audio source.

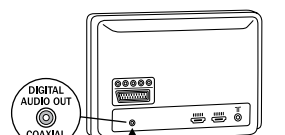

C LISTENING TO YOUR RESOUND TV STREAMER 2

After setting up and pairing, you can listen to streamed audio from the ReSound TV Streamer 2 as follows.

With your TV, sound system or computer turned on ...

EITHER: Press and hold the push button on your hearing aids for 3 seconds.

This must be done separately on both hearing aids, in order to hear streamed audio in both ears, unless your hearing aids have been set up with device-to-device synchronization;

OR: If you have a ReSound Remote Control (optional), simply press the streaming button on this ReSound Remote Control.



Streaming button

Once the ReSound TV Streamer 2 has been connected, an audible melody will play in your hearing aids and will be followed by the streamed audio.

When using the optional ReSound Remote Control, the display will show that it is searching, and then indicate that it is connected to the ReSound TV Streamer 2.

TO STOP LISTENING TO STREAMED AUDIO

EITHER: Shortly press and release the push button on your hearing aid;

OR: Press the “P” button on the remote control.

TIPS AND SUPPORT

For more details about the ReSound TV Streamer 2, visit:
<https://gs.resound.com/en-us/support/support-accessories>
